


# VBM Baby-Cap

VBM developed a special Baby-Cap which does not require the use of adhesive tape on the babies face during the fixation of the Endotracheal Tube or probes and tubes. The cap is made from soft cotton material and therefore very comfortable for the baby. Different sizes are available from the premature baby to the newborn and infant.

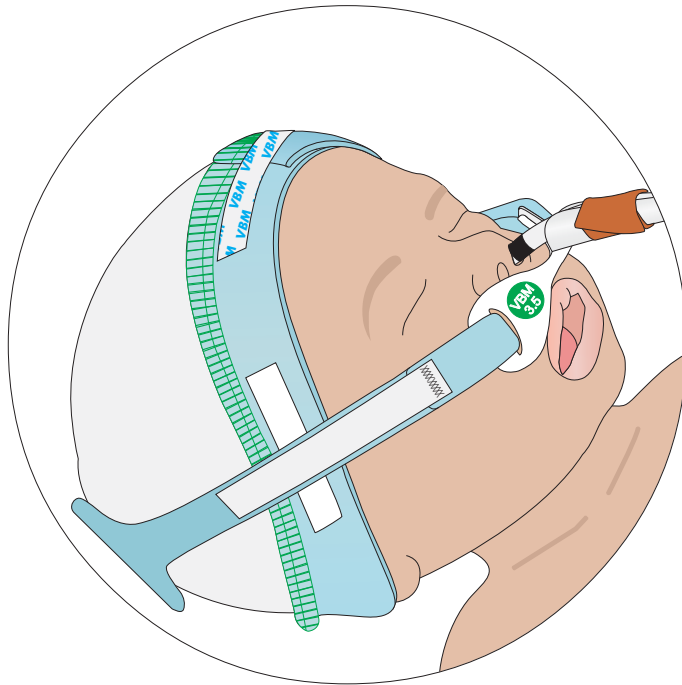



- **Baby-Cap**  
Cap with fixation system for probes and tubes
- **Baby-Cap with Endofix**  
Cap with fixation system for ET-Tubes, probes and tubes

**The Baby-Cap offers a wide variety of adjusting possibilities:**


- 6 sizes are available. Each size can be further adjusted with the velcro® strip
- The necktape is variable in length and positioning
- Flange in different sizes for suitable ET-Tube

## Product features

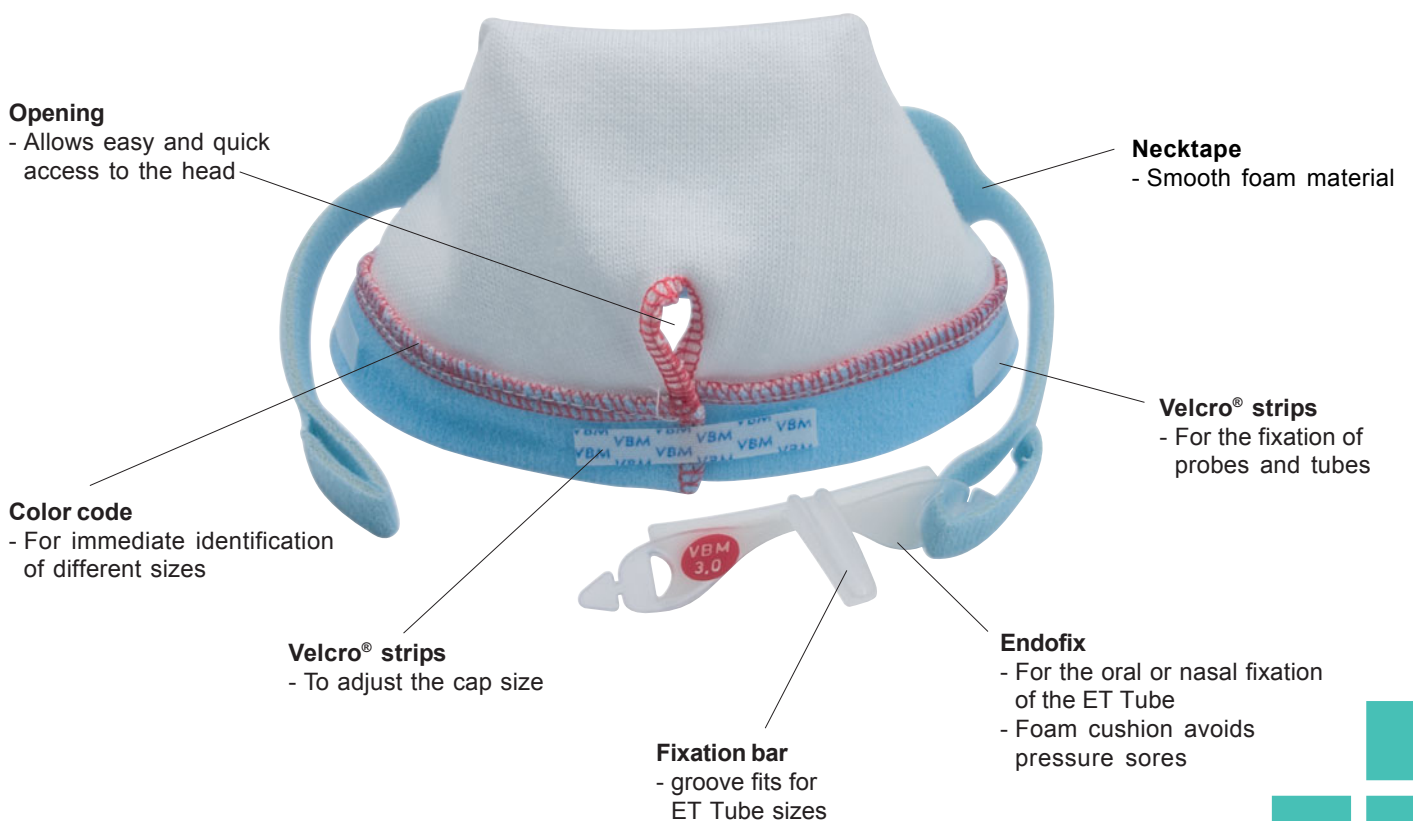


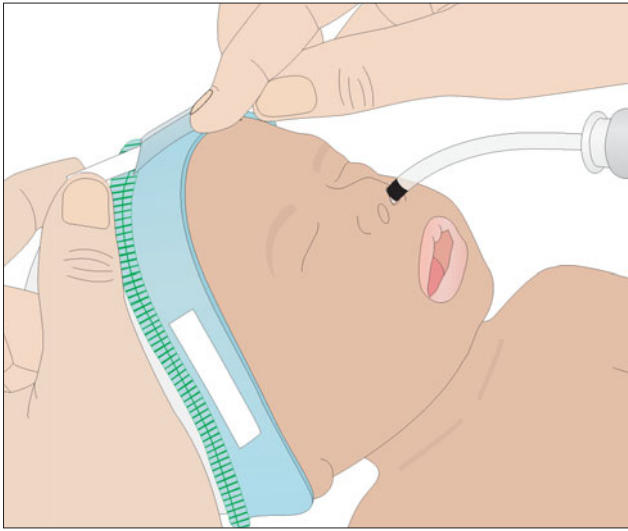


Adjust cap according to the head size. Tube is intubated nasally.

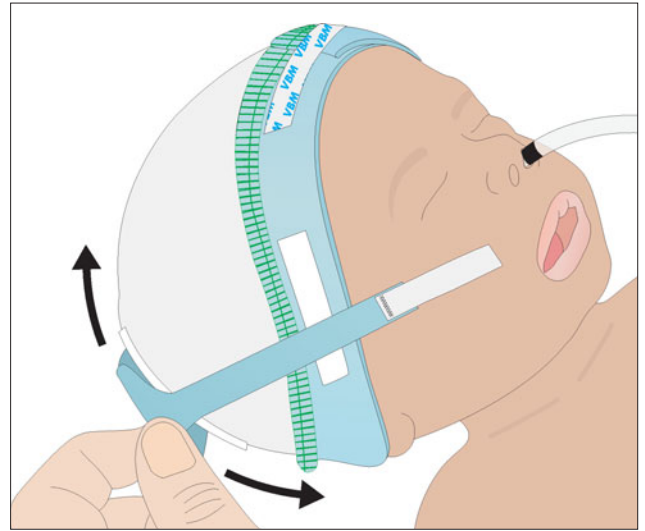

For a proper fixation the necktape can be positioned and fixed individually, from a different angle for nasal or oral intubated tube.

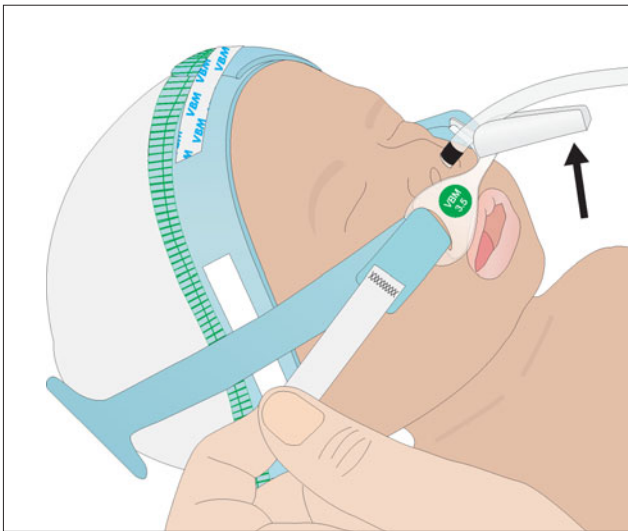

Clip tube into groove of the fixation bar and secure it with adhesive tape. Pull the necktape tight and fix the velcro fastening.

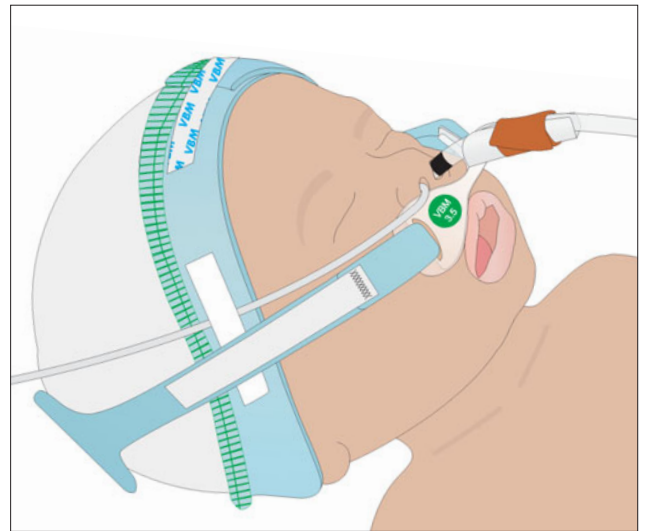

Additional probes and tubes (for example feeding tube) can also be fixed on the cap.

## Order information

Size	Color code	Head size	Baby-Cap		Baby-Cap with Endfix		Box	
			REF	REF	for ET Tube			
0	white	19-22 cm	40-1900-000	40-1905-000	2.0 - 2.5	5	Size 0 and 1 with small flange	
1	orange	23-28 cm	40-1900-001	40-1905-001	2.0 - 2.5	5		
2	yellow	28-32 cm	40-1900-002	40-1905-002	2.5	5	Size 2 to 5 with large flange	
3	red	33-38 cm	40-1900-003	40-1905-003	3.0	5		
4	green	38-43 cm	40-1900-004	40-1905-004	3.5	5		
5	blue	43-50 cm	40-1900-005	40-1905-005	4.0	5		

**VBM** Medizintechnik GmbH

Einsteinstrasse 1 | DE-72172 Sulz a.N. | Germany

Tel.: +49 7454 / 959610 | Fax: +49 7454 / 959633 | e-mail: info@vbm-medical.de | www.vbm-medical.de

